



ELISA ENZYME LINKED IMMUNOSORBENT ASSAY

Microwell Method

CMV IgG

REF V00004

For in vitro Diagnostic Use

P r o d u c t I n s e r t

Enzyme Linked Immunosorbent Assay for the **qualitative** and **quantitative** determination of IgG Antibodies to Cytomegalovirus in human serum or plasma. It is intended as an aid in the diagnosis of possible CMV infection.



Microwell Method - 96 wells
(12 x 8-well Antigen coated Strips
Individual breakaway)

INTRODUCTION

Cytomegalovirus (CMV) is a member of the Herpes virus family which includes Herpes Simplex virus (HSV) type 1 and 2, Varicella Zoster virus (VZV) and Epstein-Barr virus (EBV). It is a ubiquitous human pathogen transmitted through saliva, sexual contact, perinatally, organ transplantation or blood transfusion.

In majority of the cases, the infection remains asymptomatic. However, CMV infection can cause serious illness in newborns and immunosuppressed individuals such as patients with AIDS, HIV, cancer or patients that received organ transplants.¹

During immunosuppressive therapy, a reactivation of the latent virus or primary infection occurs frequently. For most newborns, CMV infections can be acquired before birth, during birth and later in life. The infection may cause severe congenital abnormalities such as microcephaly, motor disability, and mental retardation.^{2, 3, 4}

Therefore, determining primary maternal infections and distinguishing primary from latent infection is of great importance. The presence of IgM antibodies indicates the presence of primary infection, while presence of IgG antibodies indicates immune status of patients.

The Dialab CMV IgG ELISA Test Kit is an immunoassay for the qualitative and quantitative detection of the presence of IgG antibodies to CMV in serum or plasma specimen. The test utilizes purified CMV antigens to selectively detect antibodies to CMV in serum or plasma.

PRINCIPLE OF THE ASSAY

The Dialab CMV IgG ELISA Test Kit is a solid phase enzyme immunoassay based on indirect principle for the qualitative and quantitative detection of IgG antibodies to CMV in human serum or plasma. The microwell plate is coated with CMV antigens. During testing, the specimen diluent and the specimens are added to the antigen coated microwell plate and then incubated. If the specimens contain IgG antibodies to CMV, it will bind to the antigens coated on the microwell plate to form immobilized antigen-CMV IgG antibody complexes. If the specimens do not contain IgG antibodies to CMV, the complexes will not be formed. After initial incubation, the microwell plate is washed to remove unbound materials. The enzyme-conjugated anti-human IgG antibodies are added to the microwell plate and then incubated. The enzyme-conjugated anti-human IgG antibodies will bind to the immobilized antigen-CMV IgG antibody complexes present. After the second incubation, the microwell plate is washed to remove unbound materials. Substrate Solution A and Substrate Solution B are added and then incubated to produce a blue color indicating the amount of CMV IgG antibodies present in the specimens. Sulfuric acid solution is added to the microwell plate to stop the reaction producing a color change from blue to yellow. The color intensity, which corresponds to the amount of CMV IgG antibodies present in the specimens, is measured with a microplate reader at 450/630-700 nm or 450 nm.

MATERIALS PROVIDED

1. **Microwell plate:** 12x 8-wells strips coated with purified CMV antigens
2. **Enzyme Conjugate:** 1 vial of 12 mL; Anti-human IgG antibody bound to peroxidase; Preservative: 0.1 % ProClin™ 300
3. **Wash Buffer conc.:** 1 vial of 50 mL; 25x conc., Tris-HCl buffer containing 0,1 % Tween 20; Preservative: 0.1 % ProClin™ 300
4. **Specimen Diluent:** 1 vial of 12 mL; Tris buffer, Preservative: 0.1 % ProClin™ 300
5. **Substrate Solution A:** 1 vial of 8 mL; Citrate-phosphate buffer containing hydrogen peroxide; Preservative: 0.1 % ProClin™ 300
6. **Substrate Solution B:** 1 vial of 8 mL; Buffer containing tetramethylbenzidine (TMB); Preservative: 0.1 % ProClin™ 300
7. **Stop Solution:** 1 vial of 8 mL; 0.5.M Sulfuric acid
8. **Calibrators A-D:** 4 vials of 1 mL each. Diluted human serum containing the following amounts of CMV IgG antibodies; Preservative: 0.1 % ProClin™ 300:
 - 1) Calibrator A - 0 IU/mL
 - 2) Calibrator B -15 IU/mL
 - 3) Calibrator C - 60 IU/mL
 - 4) Calibrator D - 150 IU/mL

A human serum sample demonstrating high levels of anti-CMV IgG activity was defined as containing 150 units of CMV IgG antibody per mL (U/mL).The calibrators for the CMV IgG ELISA are manufactured by dilution and referenced to this standard.
9. **Plate sealer**
10. **Package Insert**

MATERIALS REQUIRED BUT NOT PROVIDED

- Freshly distilled or deionized water
- Sodium hypochlorite solution for decontamination
- Absorbent paper or paper towel
- Water bath or incubator capable of maintaining 37°C ± 2°C
- Calibrated automatic or manual microwell plate washer capable of aspirating and dispensing 350 µL/well
- Disposable gloves
- Calibrated micropipettes with disposable tips capable of dispensing 5, 50 and 100 µL
- Graduated cylinders for wash buffer dilution
- Vortex mixer for specimen mixing (optional)
- Timer
- Disposable reagent reservoirs
- Calibrated microplate reader capable of reading at 450 nm with a 630-700 nm reference filter, or reading at 450 nm without a reference filter
- Automated processor (optional)

PRECAUTIONS

- For professional in vitro diagnostic use only. Do not use after expiration date.
- Do not mix reagents from other kits with different lot numbers.
- Avoid cross contamination between reagents to ensure valid test results.
- Follow the wash procedure to ensure optimum assay performance.

- Use Plate Sealer to cover microwell plate during incubation to minimize evaporation.
- Use a new pipet tip for each specimen assayed.
- Ensure that the bottom of the plate is clean and dry and that no bubbles are present on the surface of the liquid before reading the plate. Do not allow wells to dry out during the assay procedure.
- Do not touch the bottom of the wells with pipette tips. Do not touch the bottom of the microwell plate with fingertips.
- Do not allow sodium hypochlorite fumes from chlorine bleach or other sources to contact the microwell plate during the assay as the color reaction may be inhibited.
- All equipment should be used with care, calibrated regularly and maintained following the equipment manufacturer's instructions.
- Calibrators A-D, Enzyme Conjugate, Sample Diluent, Substrate Solution A, Substrate Solution B, Wash Buffer:

Above reagents contain 0.1 % ProClin™ 300 as a preservative, which is classified as below:



Warning

- | | |
|------------|---|
| H317: | May cause an allergic skin reaction. |
| P261: | Avoid breathing dust/fume/gas/vapours/spray. |
| P272: | Contaminated work clothing should not be allowed out of the workplace. |
| P280: | Wear protective gloves/protective clothing/eye protection/face protection. |
| P302+P352: | If on skin: wash with plenty of soap and water. |
| P333+P313: | If skin irritation or rash occurs: Get medical advice/attention. |
| P362+P364: | Take off contaminated clothing and wash it before reuse. |
| P501: | Dispose of contents and container in accordance to local, regional, national and international regulations. |

HEALTH AND SAFETY INFORMATION

- Some components of this kit contain human blood derivatives, which were found to be non-reactive for the HIV-1/HIV-2/HIV-O, Syphilis and HCV antibodies, as well as HBsAg. No known test method can offer complete assurance that products derived from human blood will not transmit infectious agents. Therefore, all blood derivatives should be considered potentially infectious. It is recommended that these reagents and human specimens be handled using established good laboratory working practices.
- Wear disposable gloves and other protective clothing such as laboratory coats and eye protection while handling kit reagents and specimens. Wash hands thoroughly when finished.
- ProClin 300™ is included as a preservative in the Conjugate, Concentrated Wash Buffer, Specimen Diluent, Substrate Solution and Calibrators. Avoid any contact with skin or eyes.
- Do not eat, drink or smoke in the area where the specimens or kits are handled. Do not mouth pipette.
- Avoid any contact of the Substrate Solution A, Substrate Solution B, and Stop Solution with skin or mucosa. The Stop Solution contains 0.5 M sulfuric acid which is a strong acid. If spills occur, wipe immediately with large amounts of water. If the acid contacts the skin or eyes, flush with large amounts of water and seek medical attention.
- Non-disposable apparatus should be sterilized after use. The preferred method is to autoclave for one hour at 121°C. Disposables should be autoclaved or incinerated. Do not autoclave materials containing sodium hypochlorite.

- Handle and dispose all specimens and materials used to perform the test as if they contained infectious agents. Observe established precautions against microbiological hazards throughout all the procedures and follow the standard procedures for proper disposal of specimens.
- Observe Good Laboratory Practices when handling chemicals and potentially infectious material. Discard all contaminated material, specimens and reagents of human origin after proper decontamination and by following local, state and federal regulations.
- Neutralized acids and other liquids should be decontaminated by adding sufficient volume of sodium hypochlorite to obtain a final concentration of at least 1.0%. A 30 minute exposure to a 1.0% sodium hypochlorite may be necessary to ensure effective decontamination.

STORAGE AND STABILITY OF THE KIT

- Unopened test kits should be stored at 2-8°C upon receipt. All reagents are stable through the expiration date printed on the box if stored between 2-8°C. Once opened, all reagents are stable for up to 3 months after the first opening date if stored between 2-8°C. Return reagents to 2-8°C immediately after use.
- Allow the sealed pouch to reach room temperature before opening the pouch and removing the required number of strips to prevent condensation of the microwell plate. The remaining unused strips should be stored in the original resealable pouch at 2-8°C and can be used within 3 month of the opening date. Return the remaining unused strips and supplied desiccant to the original resealable pouch, firmly press the seal closure to seal the pouch completely and immediately store at 2-8°C.
- Concentrated Wash Buffer may be stored at room temperature to avoid crystallization. If crystals are present, warm up the solution at 37°C. Working Wash Buffer is stable for 2 weeks at room temperature.
- Do not expose reagents especially the Substrate Solution to strong light or hypochlorite fumes during storage or incubation steps.
- Do not store Stop Solution in a shallow dish or return it the original bottle after use.

SPECIMEN COLLECTION AND PREPARATION

- The Dialab CMV IgG ELISA Test Kit can be performed using only human serum or plasma collected from venipuncture whole blood.
- EDTA, sodium heparin, and ACD collection tubes may be used to collect venipuncture whole blood and plasma specimens. The preservative sodium azide inactivates horseradish peroxidase and may lead to erroneous results.
- Separate serum or plasma from blood as soon as possible to avoid hemolysis. Grossly hemolytic, lipidic or turbid samples should not be used. Specimen with extensive particulate should be clarified by centrifugation prior to use. Do not use specimens with fibrin particles or contaminated with microbial growth.
- Serum and plasma specimens may be stored at 2-8°C for up to 7 days prior to assaying. For long term storage, specimens should be kept frozen below -20°C.
- Bring specimens to room temperature prior to testing. Frozen specimens must be completely thawed and mixed well prior to testing. Specimens should not be frozen and thawed repeatedly.
- If specimens are to be shipped, they should be packed in compliance with local regulations covering the transportation of etiologic agents.

REAGENTS PREPARATION

Allow reagents and specimens to reach room temperature (15-30°C) prior to testing. The procedure must be strictly followed. Assay must proceed to completion within time limits. Arrange the calibrators so that well A1 is the Blank well. From well A1, arrange the calibration in a horizontal or vertical configuration. The procedure below assigns specific wells arranged in a vertical configuration. Configuration may depend on software.

WASH BUFFER:

Prepare Working Wash Buffer by diluting the Concentrated Wash Buffer 1:25. Pour the contents of the bottle in a graduated cylinder and fill it with freshly distilled or deionized water to 1250 mL. It is stable for 2 weeks at 15-30°C.

Note: If crystals are present in the Concentrated Wash Buffer, warm it up at 37°C until all crystals dissolve.

ASSAY PROCEDURE

1. Leave A1 as Blank well.
2. Add 100 µL of Calibrator 1 in wells B1 and C1. (Yellow Reagent)
Add 100 µL of Calibrator 2 in wells D1 and E1. (Blue Reagent)
Add 100 µL of Calibrator 3 in wells F1 and G1. (Blue Reagent)
Add 100 µL of Calibrator 4 in wells H1 and A2. (Blue Reagent)
3. Add 100 µL of Specimen Diluent to assigned wells starting at B2. The color of Specimen Diluent is green.
Add 5 µL of specimen to assigned wells starting at B2. Then a color change from green to blue will occur to verify that the specimen has been added.
Remove unused strips from the microwell plate, and store in the original resealable pouch at 2-8°C.
4. Mix gently by swirling the microwell plate on a flat bench for 30 seconds.
Cover the microwell plate with the Plate Sealer and incubate in a water bath or an incubator at 37°C ± 2°C for 30 minutes ± 2 minutes.
5. Remove the Plate Sealer.
Wash each well 5 times with 350 µL of Working Wash Buffer per well, then remove the liquid.
Turn the microwell plate upside down on absorbent tissue for a few seconds.
Ensure that all wells have been completely washed and dried.
Note: Improper washing may cause false positive results.
6. Add 100 µL of Conjugate to each well except for the Blank well. The color of Conjugate is red.
7. Cover the microplate plate with the Plate Sealer and incubate in a water bath or an incubator at 37°C ± 2°C for 30 minutes ± 2 minutes.
8. Repeat Step 5.
9. Add 50 µL of Substrate Solution A to each well. (Clear Reagent)
Add 50 µL of Substrate Solution B to each well. (Clear Reagent)
Then a blue color should develop in wells containing Positive specimens.
10. Mix gently then cover microwell plate with Plate Sealer and incubate in a water bath or incubator at 37°C ± 2°C for 10 minutes ± 1 minute.
11. Remove the Plate Sealer.
Add 50 µL of Stop Solution to each well. (Clear Reagent)
Then a yellow color should develop in wells containing Positive specimens.
12. Read at 450/630-700 nm in 30 minutes.
Note: Microwell plate can also be read at 450 nm, but it is strongly recommended to read it at 450/630-700 nm for better results.

ASSAY SCHEME

1. Prepare the Working Wash Buffer by diluting the Wash Buffer concentrate 1:25.
2. Follow this scheme:

REAGENTS	A1 BLANK	CALIBRATORS	SAMPLE
Calibrators	-	100 µL	-
Sample Diluent	-	-	100 µL
Sample	-	-	5 µL
Cover strips with adhesive film.			
Incubate 30 min. at +37°C.			
Peel out the adhesive film and aspirate the reaction solution from all wells.			
Wash 5 times with 350 µL of diluted Wash Buffer, carefully aspirating off the remaining liquid.			
Enzyme Conjugate	-	100 µL	100 µL
Cover strips with adhesive film.			
Incubate 30 min. at +37°C.			
Peel out the adhesive film and aspirate the reaction solution from all wells.			
Wash 5 times with 350 µL of diluted Wash Buffer, carefully aspirating off the remaining liquid.			
Substrate Solution A	50 µL	50 µL	50 µL
Substrate Solution B	50 µL	50 µL	50 µL
Cover strips with a new adhesive film.			
Incubate 10 min. at +37°C., protected from light.			
Stop solution	50 µL	50 µL	50 µL
Read the absorbance of each well against A1 blanking-well at 450 nm and 630-700 nm in 30 min.			

AUTOMATED PROCESSING

Automatic ELISA microplate processors may be used to perform the assay after validating the results to ensure they are equivalent to those obtained using the manual method for the same specimens. Incubation times may vary depending on the processors used but do not program less incubation times than the procedure listed above. When automatic ELISA microplate processors are used, periodic validation is recommended to ensure proper results.

VALIDATION REQUIREMENTS AND QUALITY CONTROL

1. Calculate the Mean Absorbance of Calibrators 1-4 by referring to the table below.

Example of Calibrator 2 Calculation

Item	Absorbance
Calibrator 2: Well D1	0.268
Calibrator 2: Well E1	0.254
Total Absorbance of Calibrator 2	$0.268 + 0.254 = 0.522$
Mean Absorbance of Calibrator 2	$0.522/2 = 0.261$

2. Check the validation requirements below to determine if the test results are valid.

Item	Validation Requirements
Blank Well	Blank Absorbance should be < 0.050 if read at 450/630-700 nm Note: It should be < 0.100 if read at 450 nm
Calibrator 1	Mean Absorbance after subtraction of Blank Absorbance should be < 0.150
Calibrator 2	Mean Absorbance after subtraction of Blank Absorbance should be > 0.150 and < 0.400
Calibrator 3	Mean Absorbance after subtraction of Blank Absorbance should be > Calibrator 2 and < Calibrator 4
Calibrator 4	Mean Absorbance after subtraction of Blank Absorbance should be > 1.200

NOTE: The test results are considered invalid if the above validation requirements are not met. Repeat the test or contact your local distributor.

INTERPRETATION OF RESULTS

Qualitative

Calculate the Index Value to obtain qualitative specimen results.

1. If the test is valid, obtain Cut-Off Value by subtracting the Blank Absorbance from the Mean Absorbance of Calibrator 2. See an example of Cut-Off Value calculation below.

Item	Absorbance
Blank Absorbance: Well A1	0.009
Cut-Off Value: Mean Absorbance of Calibrator 2 – Blank Absorbance	$0.261 - 0.009 = 0.252$


2. Calculate the Index Value by dividing the Specimen Absorbance by the Cut-Off Value, then read the results by referring to the Interpretation of Results table below.

Item	Absorbance
Specimen: Well B2	0.836
Cut-Off Value	0.252
Index Value: Specimen/Cut-Off Value	$0.836/0.252 = 3.317$

Quantitative

Draw the calibration curve and obtain quantitative specimen results.

1. Subtract the Blank Absorbance from the Mean Absorbance of each Calibrator, then plot them on the Y-axis against their concentration in U/mL on the X-axis on a linear graph paper and draw the calibration curve. Draw the best fitted line through data points to obtain a standard curve. Refer to an example of the calibration curve at right.

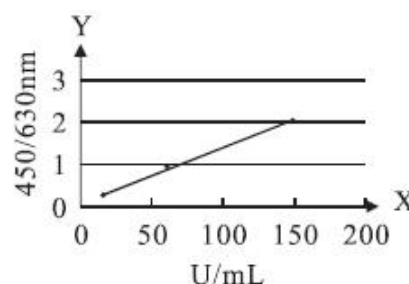

NOTE: Do not use the calibration curve at right to make any calculation. A calibration curve must be performed for each run.

2. Obtain quantitative specimen results from their absorbance by using the calibration curve.

NOTE: For Equivocal results, the specimens should be re-tested in duplicate and calculate the average value to make judgement. Specimens that are repeatedly Equivocal after re-test, should be tested using an alternate method. If the results remain Equivocal, collect a new specimen in two weeks. If the new specimen is positive, the specimen is presumed to be positive.

Interpretation of Results – Qualitative and Quantitative

Results	Qualitative	Quantitative
	Index Value	Concentration
Negative	< 0.9	$< 13.5 \text{ U/mL}$
Positive	> 1.1	$\geq 16.5 \text{ U/mL}$
Equivocal*	$\geq 0.9 \text{ and } \leq 1.1$	$13.5 - 16.5 \text{ U/mL}$

*NOTE: For Equivocal results, the specimen should be retested. Specimens that are repeatedly Equivocal after retest should be confirmed using an alternate method. If the results remain Equivocal, collect a new specimen in two weeks. If the new specimen is Positive, the specimen is presumed to be Positive.

LIMITATIONS

1. The Dialab CMV IgG ELISA Test Kit is used for the detection of IgG antibodies to CMV in human serum or plasma. Diagnosis of an infectious disease should not be established based on a single test result. Further testing, including confirmatory testing, should be performed before a specimen is considered positive. A negative test result does not exclude the possibility of exposure. Specimens Containing precipitate may give inconsistent test results.
2. As with all diagnostic tests, all results must be interpreted together with other clinical information the physician.
3. As with other sensitive immunoassays, there is the possibility that the positive result cannot be repeated due to inadequate washing from the initial test. The results may be affected due to procedural or instrument error.

PERFORMANCE CHARACTERISTICS**Sensitivity and Specificity**

The Dialab CMV IgG ELISA Test Kit has correctly identified specimens of a mixed titer performance panel and has been compared to a leading commercial Dialab CMV IgG ELISA test using clinical specimens. The results show that the clinical sensitivity of the Dialab CMV IgG ELISA Test Kit is 98.0%, and the clinical specificity is 98.3%.

CMV IgG ELISA vs. Other ELISA

Method		Other ELISA		Total Results
CMV IgG ELISA	Results	Positive	Negative	
	Positive	100	1	101
	Negative	2	58	60
Total Results		102	59	161

Clinical Sensitivity: 98.0% (93.1-99.8%)*

Clinical Specificity: 98.3% (90.9-100.0%)*

Overall Agreement: 98.1% (94.7-99.6%)*

*95% Confidence Interval

REPRODUCIBILITY

Intra-Assay: Within-run precision has been determined by using 15 replicates of three specimens: a low positive, a medium positive, and a high positive.

Inter-Assay: Between-run precision has been determined by 3 independent assays on the same three specimens: a low positive, a medium positive, and a high positive. Three different lots of the Dialab CMV IgG ELISA Test Kit have been tested using these specimens over a 5-day period.

Specimen	Intra-Assay			Inter-Assay		
	Mean Absorbance / Cut-Off	Standard Deviation	Coefficient of Variation (%)	Mean Absorbance / Cut-Off	Standard Deviation	Coefficient of Variation (%)
1	1.752	0.128	7.306	1.838	0.120	6.529
2	4.431	0.349	7.876	4.439	0.290	6.533
3	9.041	0.723	7.997	9.017	0.774	8.584

INTERFERENCES

Interferences are not observed up to a concentration of 1 mg/mL Acetaminophen, 0.2 mg/mL Gentistic Acid, 0.1 mg/mL Ascorbic Acid, 0.1 mg/mL Acetosalisilyc Acid, 0.1 mg/mL Caffeine, 0.6 mg/mL Oxalic Acid, 2 mg/mL Bilirubin, 2 mg/mL Hemoglobin, 10% H₂O₂, 1 % Methanol and 1% Ethanol. Rheumatoid factors do not interfere with the test.

Cross-reactivity is not observed in Syphilis, HBsAg, HIV, HCV, HSV IgG, Toxoplasma IgG and Rubella IgG positive specimens.

REFERENCES

1. Hodinka, RL, and Friedman, HM. Human Cytomegalovirus. In: Manual of Clinical Microbiology 6th Edition (1995) 884-894.
2. Hanshaw, JB, Scheiner, AP, Moxley, AW, Gaev, L, Abel, V, and Scheiner, B. School Failure and Deafness after "Silent" Congenital Cytomegalovirus Infection. N. Engl. J. Med.(1976) 295:468-470.
3. Reynolds, DW, Stagno, S, Stubbs, KG, Dabte, AJ, Livingston, NM, Saxon, SS, Alford, CA. Inapparent Congenital Cytomegalovirus. N. Engl. J. Med. (1974) 790:291-296.
4. Stern, H. Cytomegalovirus Vaccine: Justification and Problems. In: Waterson AP (ed.) Recent Advances in Clinical Virology (1977) 117-134.

USED SYMBOLS



Manufacturer



Catalogue number



Use by: exp.



Temperature limitation



Batch code



Content



In vitro diagnostic medical device



Contains sufficient for <n> tests

ELISA Enzyme Linked Immunosorbent Assay



DIALAB Produktion und Vertrieb von chemisch – technischen Produkten
und Laborinstrumenten Gesellschaft m.b.H.
IZ-NOE Sued, Hondastrasse, Objekt M55, A-2351 Wiener Neudorf, Austria
Phone: ++43 (0) 2236 660910-0, Fax: ++43 (0) 2236 660910-30, e-mail:
office@dialab.at